



AirFit™ N20
Nasal mask



The compact nasal mask designed to easily fit everyone

Every patient's face is different, but with our new AirFit N20, fitting a broad range of patients can be easy. This compact nasal mask gives patients a clear line of sight and accommodates their movements thanks to the InfinitySeal™ silicone cushion – made to seal robustly regardless of facial shape or size.



Easy to fit

In a ResMed international study, our new AirFit N20 fit 99.4% of all patients tested.¹ Set patients up right the first time: the silicone cushion adapts to different face shapes and sizes.

Comfortable to use

There's lightness and softness all around – from the cushion to the frame to the headgear.


Visual freedom

With its streamlined, minimal-contact fit and under-eye frame, the AirFit N20 allows patients to read or watch TV in bed.

[ResMed.com/AirFitN20](https://www.ResMed.com/AirFitN20)

Patients tested and preferred

92% of patients said they preferred wearing the AirFit N20 over the current market-leading nasal mask.²

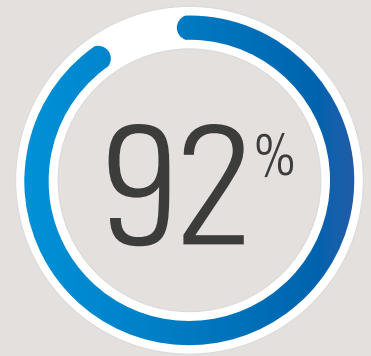

Soft and gentle

The plush headgear is specifically designed for extra softness and to help ensure patients settle into therapy quickly and comfortably.

Dependable seal

Set patients up with a reliable and effective seal thanks to the InfinitySeal silicone cushion – designed to prevent blow-outs, tolerate misfitting and seal well under variable pressures.

Extra movement

A short, flexible tube interconnects the elbow to the machine tube. It's designed to allow patients to move more easily without encountering tube drag or leakage.

Soft frame

Padding has been seamlessly incorporated into the face-side of the frame to ensure comfort at every touchpoint.

Visual freedom

Let patients see and do more in bed with an under-the-eye frame that combines all-round stability and less facial contact.

Fast fitting time


Taking the mask on and off is convenient and easy, with magnetic clips that guide the headgear to the frame in seconds.




AirFit N20 for Her

Designed specifically for women, it offers a smaller size to suit female facial contours.

In a ResMed international study,
AirFit N20 fit 99.4% of all patients.¹

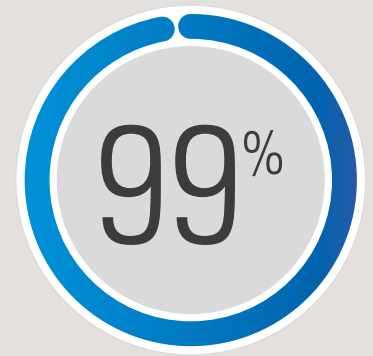

Minimal parts make the AirFit N20 easy to use and clean

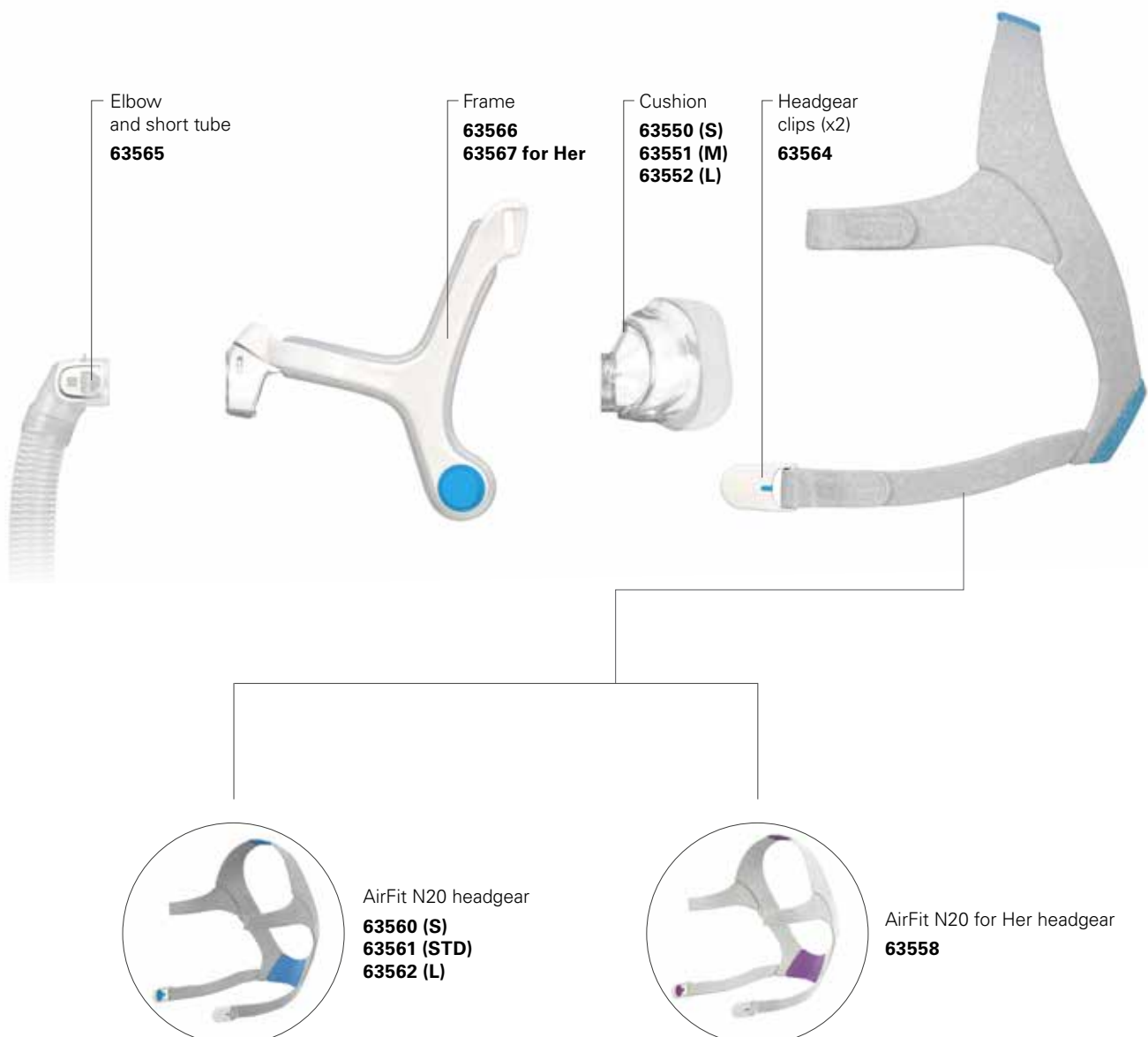


Sizing tips

AirFit N20 has three interchangeable cushion sizes - small, medium and large. The modular design of the AirFit N20 means each size cushion easily clicks into the same mask frame, so you can swap between them with ease for more flexibility.

We recommend that you refer to the mask fitting template to gauge which size is the most suitable for each of your patients.

You can download it on [ResMed.com/AirFitN20](https://www.resmed.com/AirFitN20)

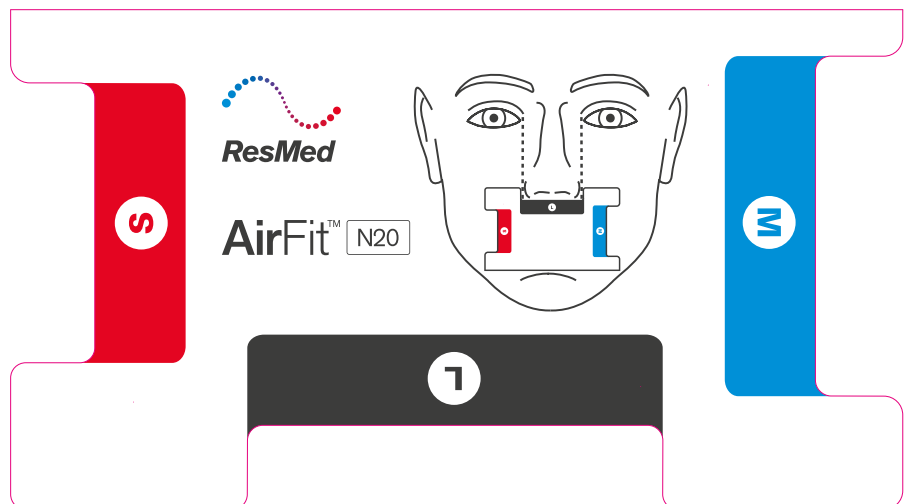

Real size template


Product codes

	APAC	ANZ	China	Japan
AirFit N20 for Her: Small	63523	63520	63528	63531
AirFit N20: Medium	63524	63521	63529	63532
AirFit N20: Large	63525	63522	63530	63533


APAC: English, Simplified Chinese, Traditional Chinese



88% of patients ranked AirFit N20 higher in comfort over the market-leading nasal mask.²

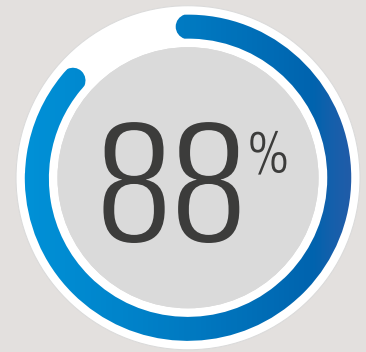

Evolving cushion technology: variable thickness designed for more comfort and stability

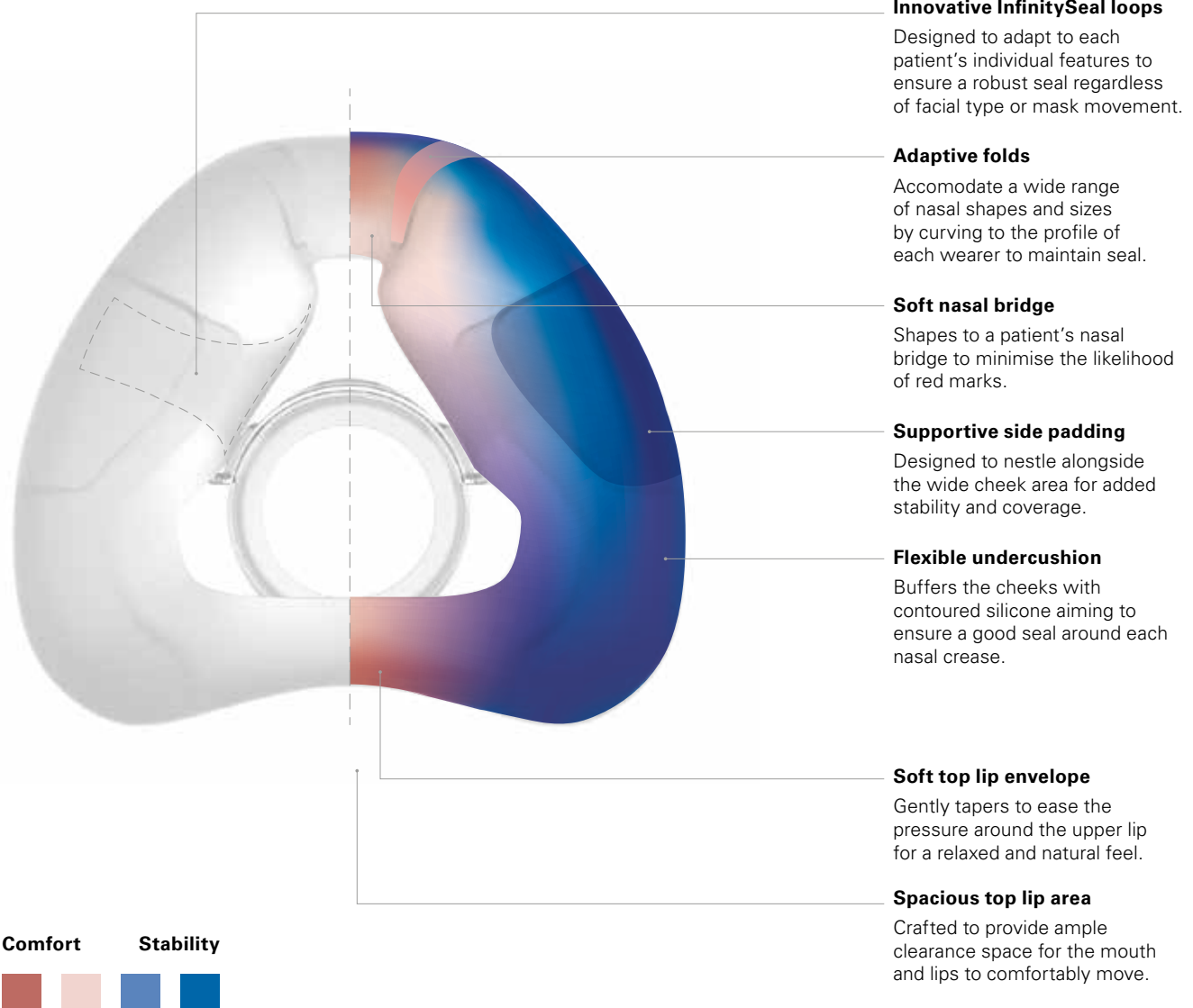



- Ensure the magnets on the lower straps are unclipped.
- If they are clipped, detach them by twisting and pulling them away from the frame.



- Pull the headgear over the head making sure the ResMed logo is facing outwards.



- Position the lower straps so they both sit below the ears.
- Bring each of the magnets up to meet their corresponding clips on the frame.



- Adjust the fastening tabs on the upper straps until the mask feels even, stable and comfortable.
- Reattach the fastening tabs.



- Repeat Step 4 with the lower straps.



- Connect the device's air tube to the elbow, then click the elbow into the front of the mask.
- Start the device.

Refer to the user guide for further instructions.



Additional options

- A patient can position the lower part of the headgear over or under their hair to suit any hairstyle preference.

1 ResMed AirFit N20 internal global fitting study of 159 existing ResMed patients, conducted 12/11/2015 ; ID A3697629.

2 ResMed internal study of 24 existing ResMed patients, conducted between 23/11/2015 - 21/12/2015 comparing the current market leading mask with AirFit N20 ; ID A3695085.