# RESMED

**Swift™ FX Nano**

**Swift™ FX Nano for Her**

**Mirage™ FX**

**Mirage™ FX for Her**

**Mirage™ SoftGel**

**Mirage Activa™ LT**

**Mirage Micro™**

**Mirage Activa™**

VENTED NASAL MASK

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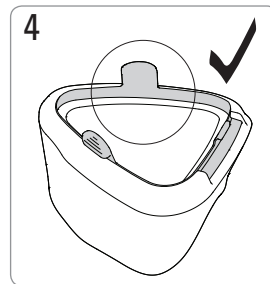
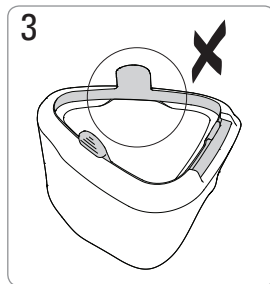
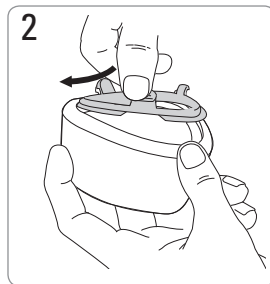
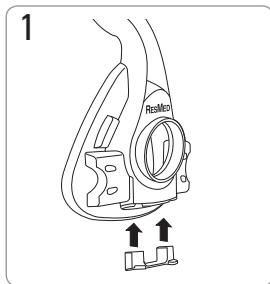
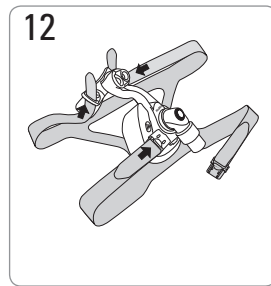
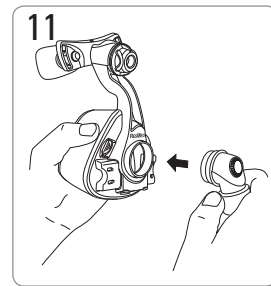
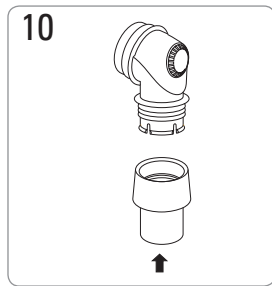
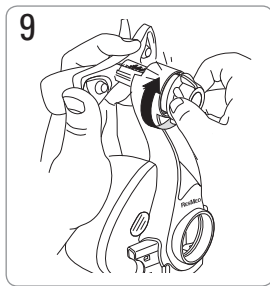
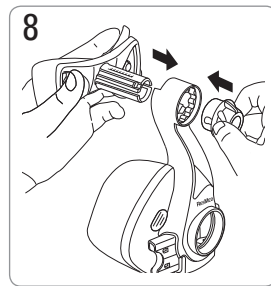
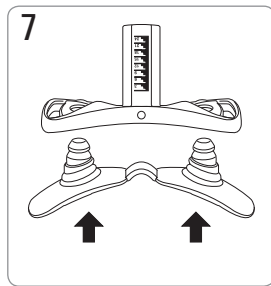
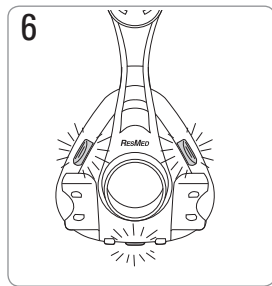
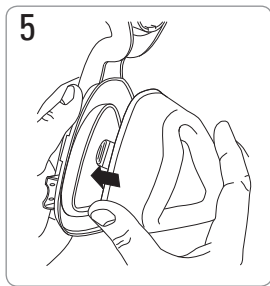
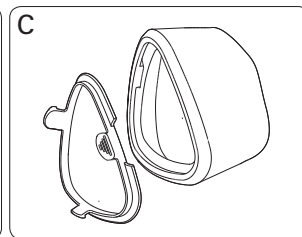
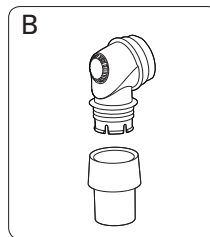
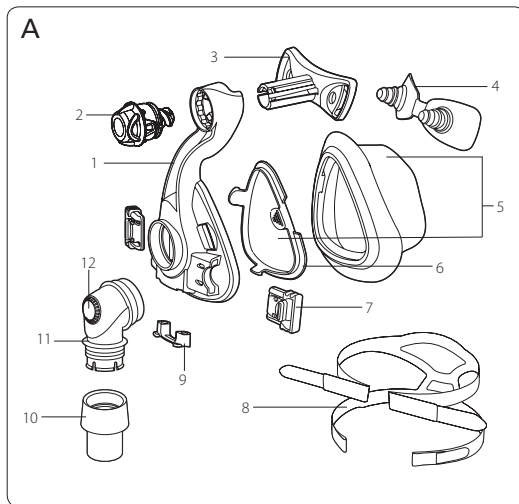
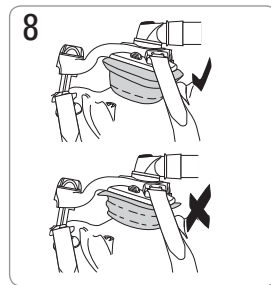
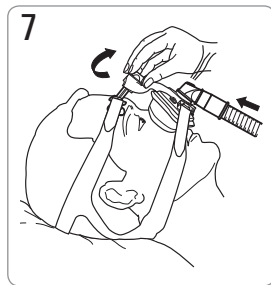
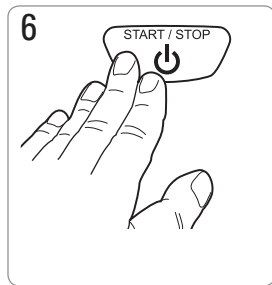
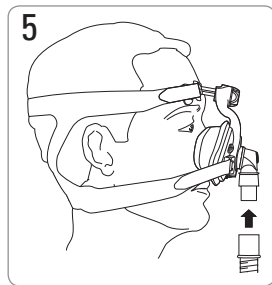
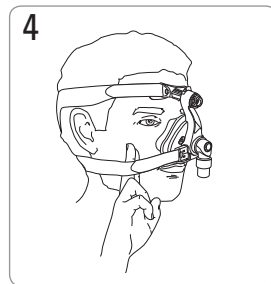
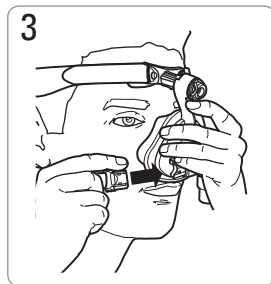
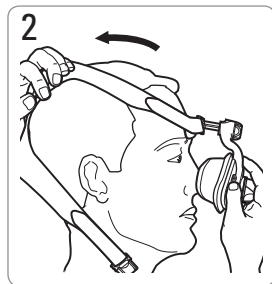
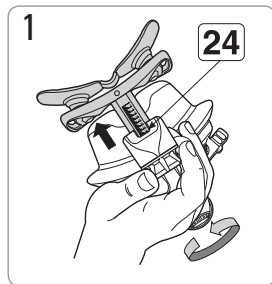
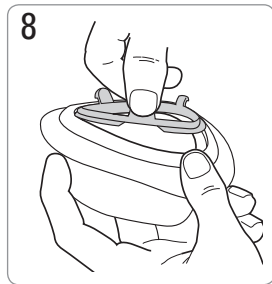
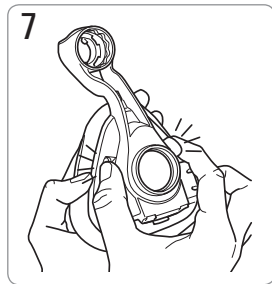
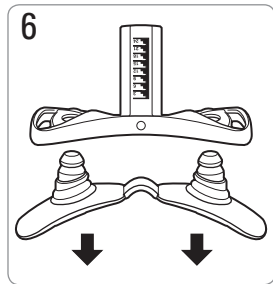
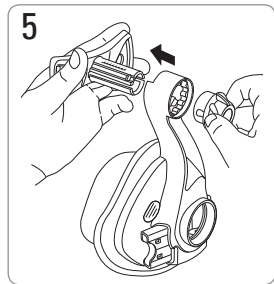
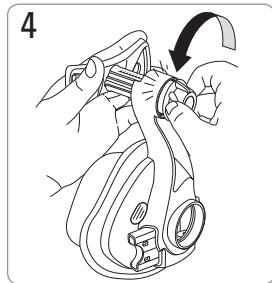
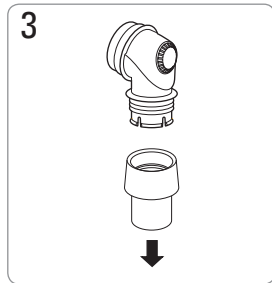
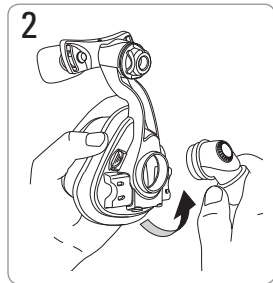
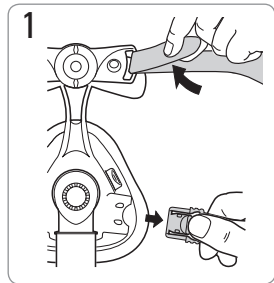
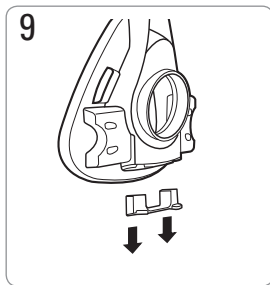
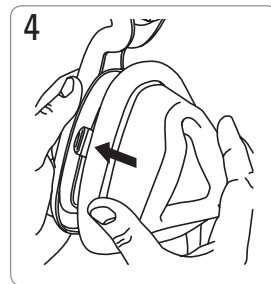
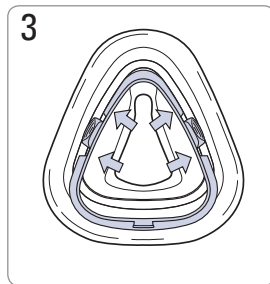
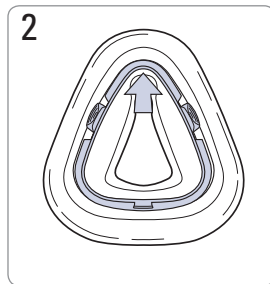
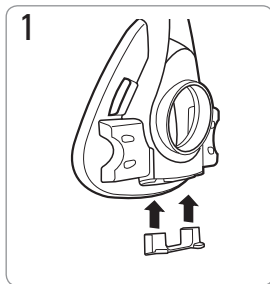
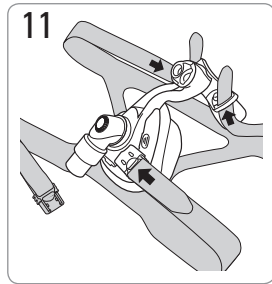
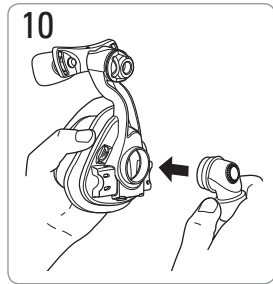
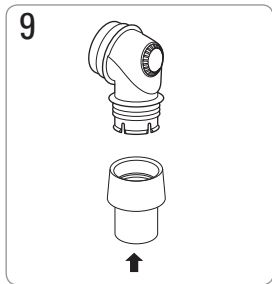
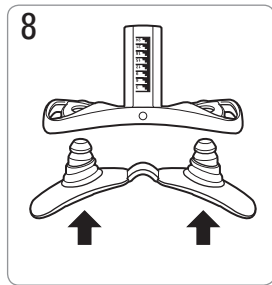
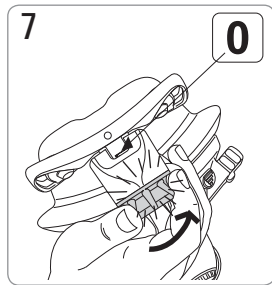
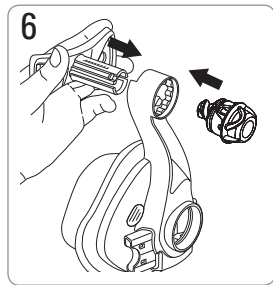
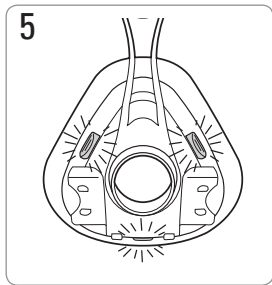
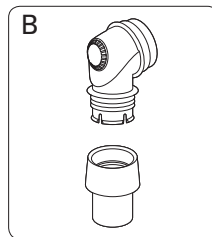
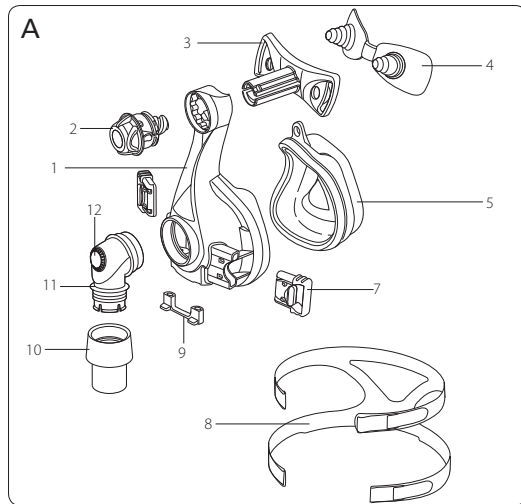
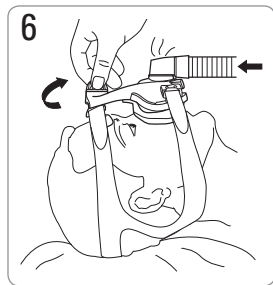
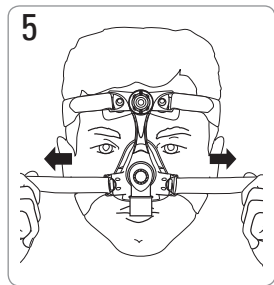
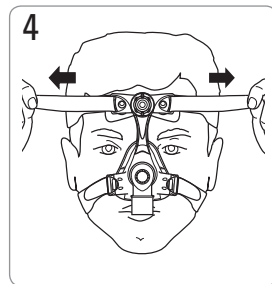
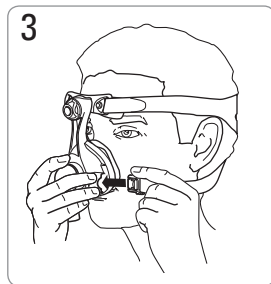
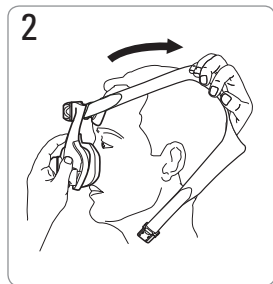
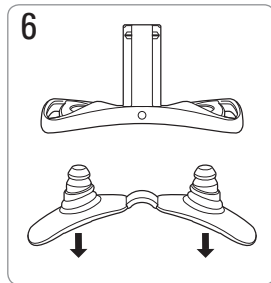
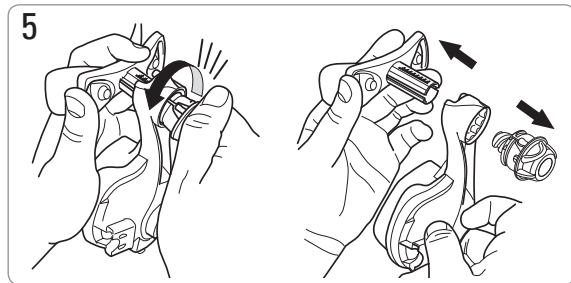
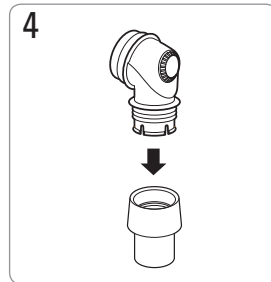
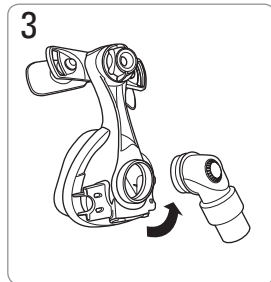
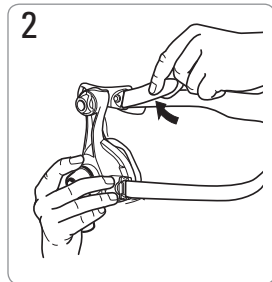
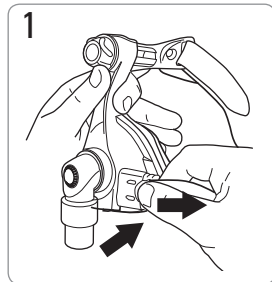
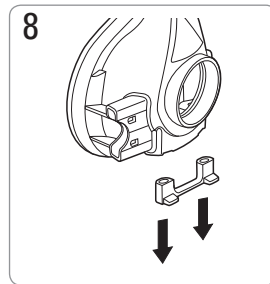
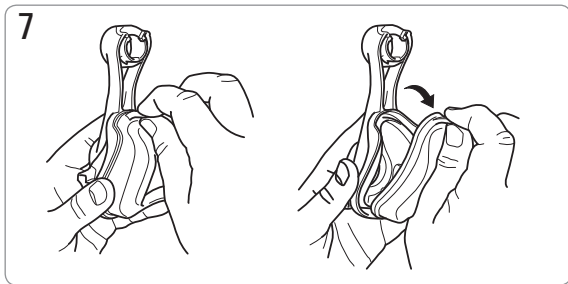
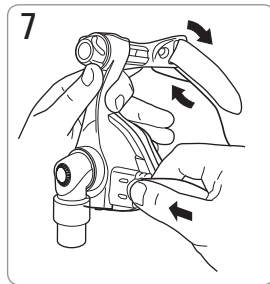
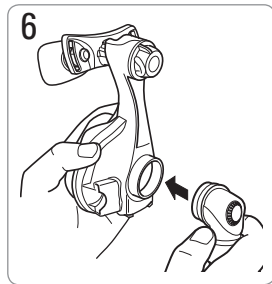
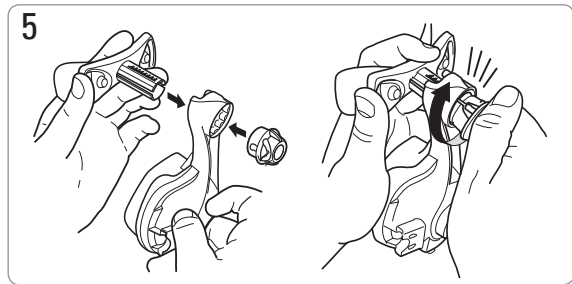
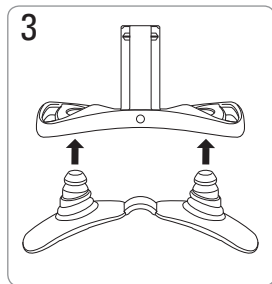
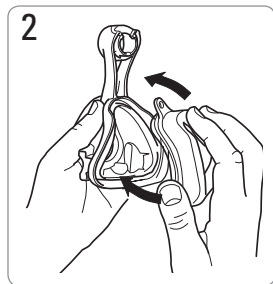
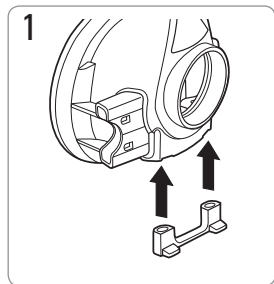
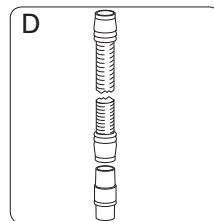
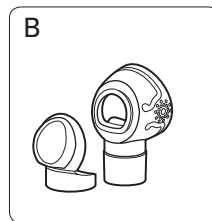
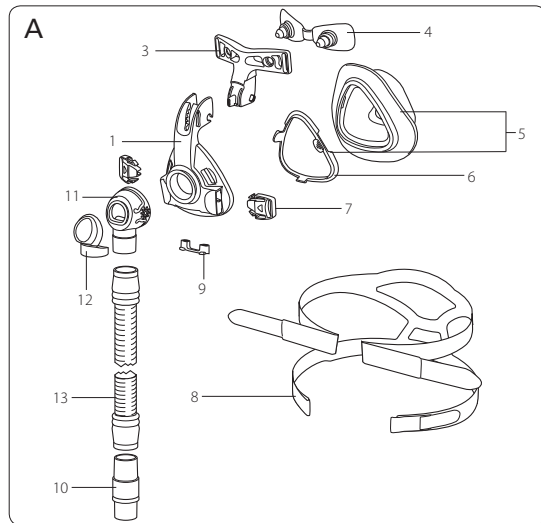
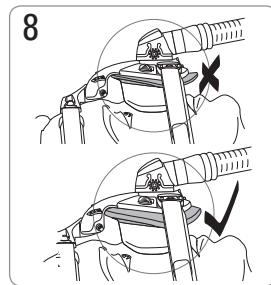
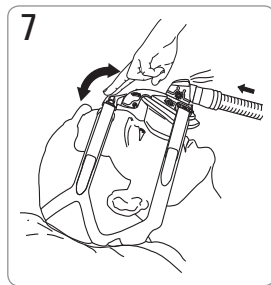
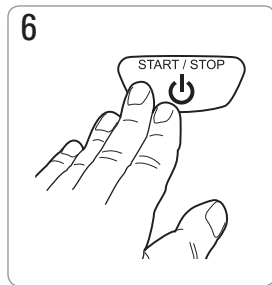
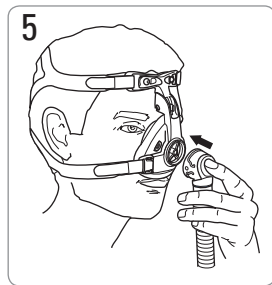
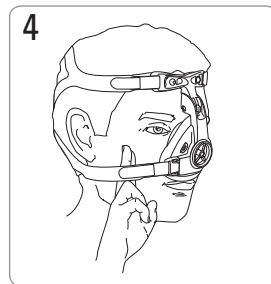
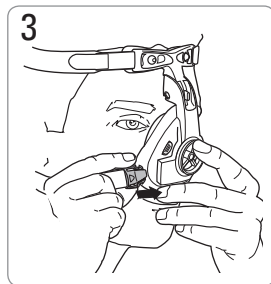
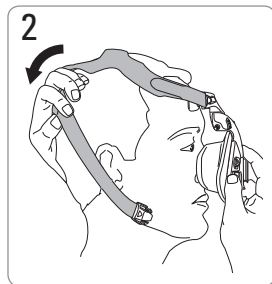
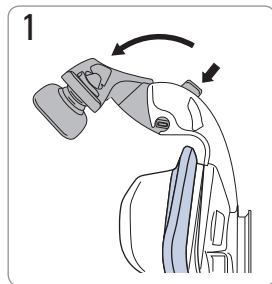
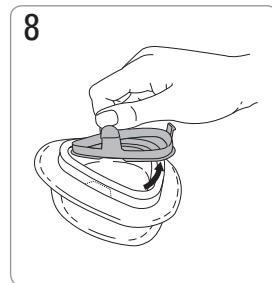
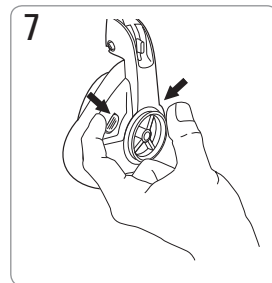
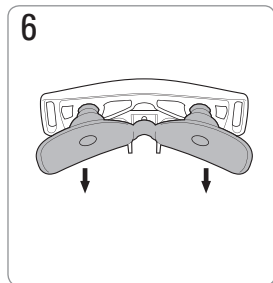
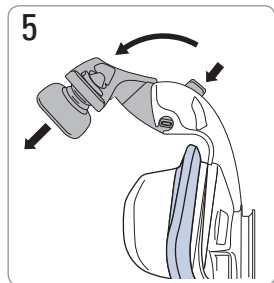
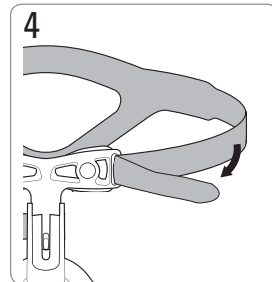
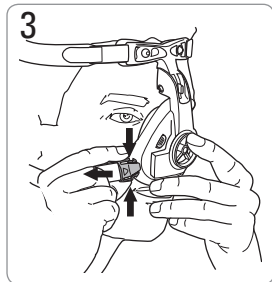
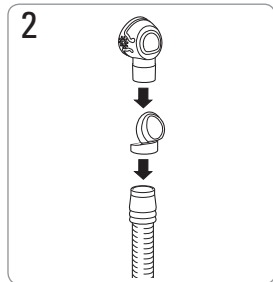
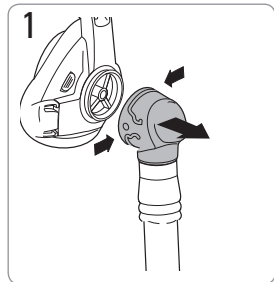
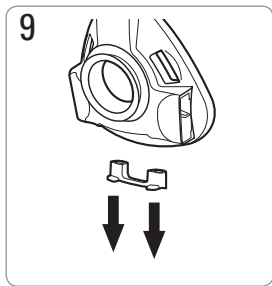
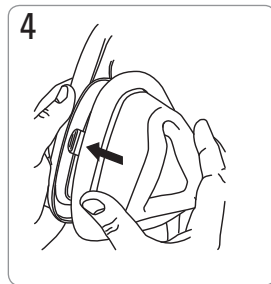
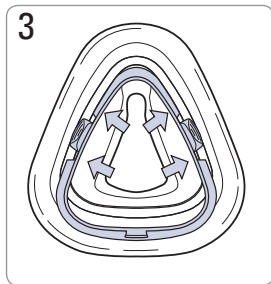
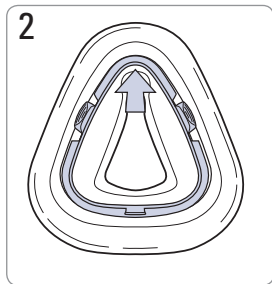
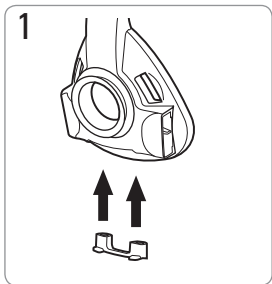
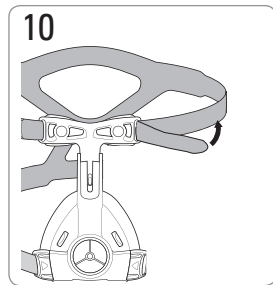
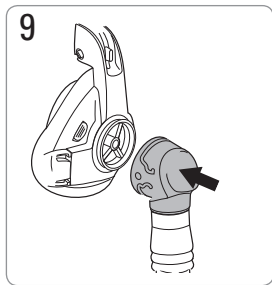
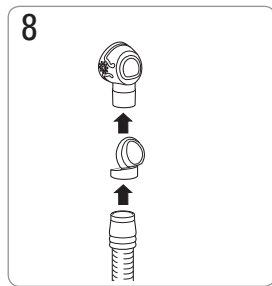
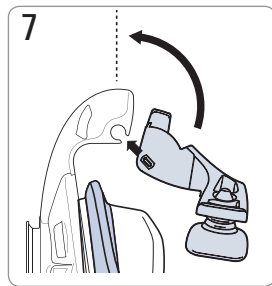
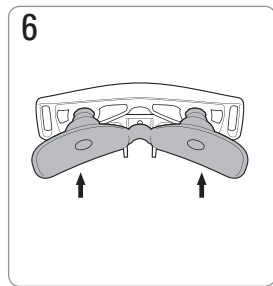
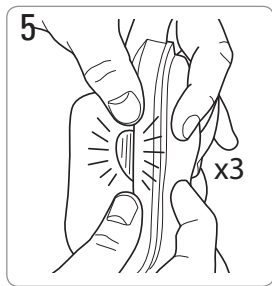
The mask channels airflow noninvasively to a patient from a continuous positive airway pressure (CPAP) or bilevel device.

The mask is:

- to be used by patients >30 kg for whom positive airway pressure has been prescribed
- intended for single-patient re-use in the home environment and multipatient re-use in the hospital/institutional environment.

### **Intended Use (Mirage SoftGel/ Mirage Activa LT/ Mirage Micro/ Mirage Activa)**

The mask directs airflow noninvasively to a user from a positive airway pressure device such as a continuous positive airway (CPAP) or bilevel system.

The mask is intended for:

- adult users (>30 kg) for whom positive airway pressure has been prescribed
- single user re-use in the home environment and multipatient re-use in the hospital/institutional environment.

### **Using the Mask**

- Refer to the Technical specifications section in this user guide for mask setting options.
- Follow the instructions provided by your physician or sleep therapist.
- To fit your mask, follow the instructions in the Fitting section.
- Do not overtighten the headgear straps. If you feel air-leaks you may need to:
  - refit or reposition your mask
  - adjust your headgear
  - check that your mask is assembled correctly
  - contact your physician or sleep therapist to have your mask refitted.
- Mask-specific user guides are available on [www.resmed.com](http://www.resmed.com) on the **Products** page under **Service & Support**. They are also available from ResMed upon request.

*The mask system does not contain latex, PVC or DEHP materials.*

### **Interchangeable Frame**

ResMed's interchangeable frame accommodates both Mirage SoftGel and Mirage Activa LT cushions.

### **Device Settings**

The mask setting options for ResMed devices are listed in the "Mask Compatibility Table" on [www.resmed.com](http://www.resmed.com) on the **Products** page under **Service & Support**.

### **Disassembly**

To disassemble your mask, follow the instructions in the Disassembly section.

### **Reassembly**

To assemble your mask, follow the instructions in the Reassembly section.

### **Cleaning the Mask at Home**

- *The mask and headgear can only be handwashed. The headgear may be washed without removing the headgear clips.*
- *Take care when cleaning to avoid damage or loss of components.*

### **Daily/After Each Use**

- To optimise mask seal, remove facial oils from the cushion before use.
- Do not soak the mask.
- Handwash the mask by rubbing gently in warm (30°C) water using mild soap or diluted dishwashing detergent.
- Rinse all components well with drinking quality water and allow them to air dry out of direct sunlight.
- If the vent or rotating parts like the swivels, ball joint or ring require cleaning, use a soft bristle brush.

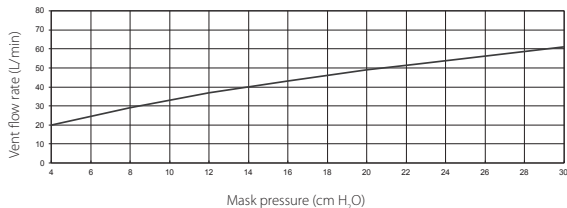
### **Weekly**

Handwash the headgear in warm (30°C) water using mild soap or diluted dishwashing detergent. Rinse well and allow it to air dry out of direct sunlight.


## Technical Specifications

The mask contains passive venting to protect against rebreathing. As a result of manufacturing variations, the vent flow rate may vary.

**Pressure-flow curve – Swift FX Nano/ Swift FX Nano for Her**

**Pressure-flow curve – Mirage FX/ Mirage FX for Her/ SoftGel/ Activa LT/ Micro/ Activa**

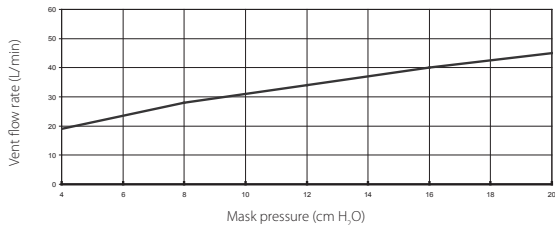	Swift FX Nano/ Swift FX Nano for Her	Mirage FX/ Mirage FX for Her	SoftGel	Activa LT	Micro	Activa*
<b>Pressure</b>	<b>Flow (L/Min)</b>					
4 cm H <sub>2</sub> O	20	19	19	19	19	19
8 cm H <sub>2</sub> O	29	28	28	28	28	28
12 cm H <sub>2</sub> O	37	34	34	34	34	34
16 cm H <sub>2</sub> O	43	40	40	40	40	40
20 cm H <sub>2</sub> O	49	45	45	45	45	45
22.5 cm H <sub>2</sub> O	52	-	-	-	-	-
30 cm H <sub>2</sub> O	61	-	-	-	-	-

**Deadspace:** Empty volume of mask to swivel. The deadspace of the mask varies according to cushion size.

125 mL (Wide)	83 mL (Wide)	145 mL (Large)	205 mL (Large Wide)	128 mL (Extra large)	311 mL (Shallow)
------------------	-----------------	-------------------	---------------------------	----------------------------	---------------------

**Therapy pressure (cm H<sub>2</sub>O)**

4 - 30	4 - 20	4 - 20	4 - 20	4 - 20	4 - 20
--------	--------	--------	--------	--------	--------

**Resistance:** Drop in pressure measured (nominal) (cm H<sub>2</sub>O)

at 50 L/min	0.5	0.2	0.3	0.1	0.1	0.1
at 100 L/min	2.0	1.0	0.9	0.7	0.7	0.5

**Sound** - DECLARED DUAL-NUMBER NOISE EMISSION VALUES in accordance with ISO 4871. The A-weighted sound power level and the A-weighted sound pressure level of the mask, at a distance of 1M, with uncertainty of 3 dBA, are shown.

Power Level	25 dBA	26 dBA	29 dBA	30 dBA	30 dBA	32 dBA
Pressure level at 1m	17 dBA	18 dBA	22 dBA	22 dBA	22 dBA	24 dBA

	Swift FX Nano/ Swift FX Nano for Her	Mirage FX/ Mirage FX for Her	SoftGel	Activa LT	Micro	Activa*
--	--	------------------------------------	---------	-----------	-------	---------

**Gross Dimensions** - Mask fully assembled - no headgear. Largest variant only shown.

Height	430 mm	138 mm	155 mm	154 mm	155 mm	425 mm
Width	114 mm	86 mm	91 mm	95 mm	91 mm	110 mm
Depth	75 mm	88 mm	118 mm	117 mm	98 mm	135 mm

\* Includes inlet tube and swivel assembly

**Mask setting options:** Select **A** if available, otherwise select **B**.

A	Pillows/ Swift	Standard	Activa	Activa	Standard	Activa
B	Mirage	Vista	Standard	Standard	Vista	Standard

**Environmental conditions** – all masks:

**Operating temperature:** +5 °C to +40 °C

**Operating humidity:** 15 % to 95 % non-condensing

**Storage and transport temperature:** -20 °C to +60 °C

**Storage and transport humidity:** up to 95 % noncondensing

## Storage

Ensure that the mask is thoroughly clean and dry before storing it for any length of time. Store the mask in a dry place out of direct sunlight.

## Disposal

The mask does not contain any hazardous substances and may be disposed of with your normal household refuse.

## Symbols

The following symbols may appear on the mask or packaging.

 Caution, consult accompanying documents;  Batch Code;  Catalogue Number;  European Authorised Representative;  Temperature limitation;  Humidity limitation;  Not made with natural rubber latex;  Manufacturer.

## Warnings

- The vent holes must be kept clear.
- As with all masks, some rebreathing may occur at low CPAP pressures.
- The mask should not be used unless the device is turned on. Once the mask is fitted, ensure the device is blowing air.
- Follow all precautions when using supplemental oxygen.
- Oxygen flow must be turned off when the CPAP or bilevel device is not operating so that unused oxygen does not accumulate within the device enclosure and create a risk of fire.
- At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration varies depending on the pressure settings, user breathing pattern, mask, point of application and leak rate.
- If any visible deterioration of a mask component is apparent (cracking, tears, etc), the mask component should be discarded and replaced.
- Do not use aromatic-based solutions or scented oils (eg, eucalyptus or essential oils), bleach, alcohol or products that smell strongly (eg, citrus) to clean any of the mask components. Residual vapours from these solutions can be inhaled and may also damage the mask, causing cracks.
- Your physician or sleep therapist is responsible for ensuring that the mask is compatible with the device. If used outside specification or if used with incompatible devices, the seal and comfort of the mask may not be effective and optimum therapy may not be achieved.
- Do not combine components from different mask systems.
- Discontinue using the mask and consult your physician or sleep therapist if you have ANY adverse reaction.
- Refer to your CPAP or bilevel device manual for details on its settings and operational information.

- This mask should be disinfected and/or sterilised when used between users.
- Remove all packaging before using the mask.
- Using a mask may cause tooth, gum or jaw soreness or aggravate an existing dental condition. If symptoms occur, consult your physician or dentist.

## Consumer Warranty

ResMed acknowledges all consumer rights granted under the EU Directive 1999/44/EG and the respective national laws within the EU for products sold within the European Union.

## Clinical Information

The mask contains passive venting to protect against rebreathing.

Technical specifications, cleaning, disinfection and sterilisation instructions are available in the mask specific user guide on [www.resmed.com](http://www.resmed.com) on the **Products** page under **Service & Support**.

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© 2015 ResMed Ltd.
Swift, Mirage, Mirage Micro and Activa are trademarks of ResMed Ltd and Swift, Mirage and Activa are registered in U.S. Patent and Trademark Office.
ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia.
ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA.
ResMed (UK) Ltd 96 Milton Park Abingdon Oxfordshire OX14 4RY UK.